

ANTERIOR TLSO FITTING PROTOCOL




Step 1. Position the panels across with the velcro facing the front



Step 2. Attach the anterior bar to the velcro on the front



Step 3. Insert the strap on the right side of the anterior panel and tighten the strap

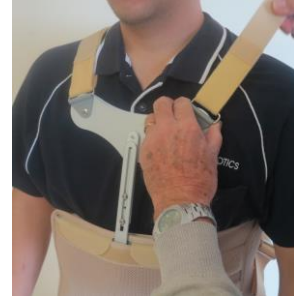

Step 4. Insert the strap on the left side of the anterior panel and tighten the strap



Step 5. Loosen the straps on the left side to the front



Step 6. Thread through the strap from the right side through the loop on the left strap



Step 7. Thread through the bottom strap from the right side to the loop on the left strap



Step 8. Tighten the straps so it is firm



Step 9. Finished

If you require any further information, please call Korthotics on (02) 8710-4183 and ask for orthotic clinical team.