

# ELASTISED TLSO FITTING PROTOCOL

Step 1. Centralized TLSO and Distal border = apex of gluteals




Step 2. Do up anterior apron



Step 3. Do up lumbar supports right and left




Step 4. Cross over shoulder straps posteriorly (as a figure 8) and through keeper

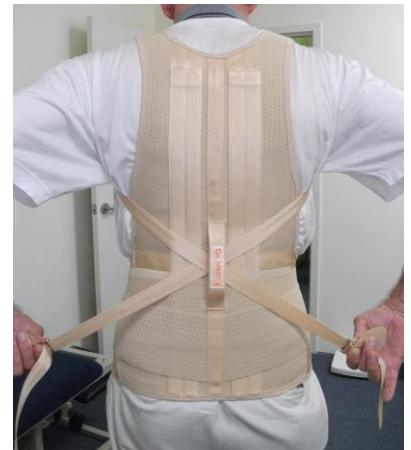

Step 5. Left strap acts on right shoulder retraction.



Step 6. Right strap acts on left shoulder retraction

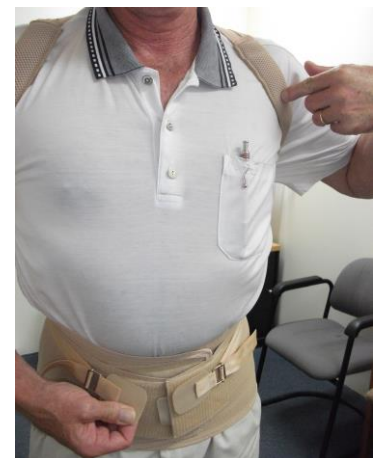

Step 7. Finished



If you require any further information, please Korthotics on (02) 8710-4183 and ask for orthotic clinical team.