


TLSO WITH BOA FITTING PROTOCOL



Step 1. Pull gently / disengage both buttons on the back panel before fitting to the patient

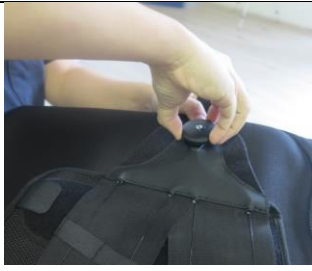

Step 2. Once the back panel is in position, position the anterior shell to the patient



Step 3. Place the 2 lateral flaps towards the front



Step 4. Thread the left strap from the back to the top of the left loop of the anterior bar and adjust the strap



Step 5. Thread the right strap from the back to the top of the right loop on the anterior bar and adjust the strap



Step 6. Push both buttons to fasten



Step 7. Turn the front buttons **clockwise** to tighten until firm



Step 8. Make sure both buttons are tightened



Step 9. Finished

If you require any further information, please call Korthotics on (02) 8710-4183 & ask for orthotic clinical team.